

# ET-1515 Back Wrap Care & Maintenance

## Wrap:

- If the wrap has become damp during use, let it dry naturally before using again.
- Do not wash the wrap in water. Do not wash the wrap in the washing machine.
- Lay the wrap out flat or hang it up to dry naturally. Do not tumble dry. The wrap may be cleaned using a lightly dampened sponge.
- Do not use bleach when washing the wrap.
- Do not dry clean your wrap. Do not dry it over anything hot (e.g. a radiator) as it contains plastic parts. Ensure the wrap is completely dry before using it again. The wrap should never be ironed.
- Always detach the lead wires from the snap and the unit. Do not pull on the lead wire cables but on the connectors attached to the ends of the cables.