Hot and Cold Therapy Gel Pack Instructions

For Hot Therapy:

Place the gel pack flat in the microwave and set the power level to high. In a 700 – 800 watt microwave, heat the gel pad no more than 50 seconds. To heat without a microwave, place the gel pack in a large bowl and pour boiling water over it. Keep the gel pad submerged in the water for a minimum of 2 minutes to have it heated sufficiently. Use rubber coated or wooden tongs to remove the gel pad and reduce the risk of puncture.

For Cold Therapy:

Place the gel pack in a freezer for a minimum of one hour before use. Keep the gel pack in the freezer longer for colder and longer therapy sessions. To be able to use your Hot & Cold Therapy Gel Pack for an emergency, keep it in a freezer when it's not in use, so it's always ready for immediate use.

Storage

Keep the gel pad away from extreme heat and open flames. Store in a dry place at room temperature.

Caution

The gel packs can reach very high and low temperatures. Wrap the gel pack in a cloth or towel before applying it to bare skin. People with known or suspected circulatory problems should not use this gel pack except under the direct supervision of a doctor. If the hot or cold therapy becomes uncomfortable, remove the gel pack for a few minutes, then re-apply.

Warning

For external use only. Harmful if swallowed. If gel solution is swallowed, drink several glasses of water and contact your poison control center immediately. Do not puncture outer bag. If the gel pack breaks and the gel comes in contact with skin of eyes, flush with water immediately and call your physician. Keep out of reach of children. Do not mix with combustible materials.

If you have any questions whatsoever regarding your iReliev® Hot & Cold Gel Pack, please contact your reseller or ExcelHealth Inc. at 855-723-2582 or visit www.iReliev.com.

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